



## XR: Manually Transfer Images to a Selected Network/PACS destination


*Definium™ 5000 and Brivo™ XR385*

This reference guide is used to manually transfer images to a Selected Network/PACS destination.

### 1. Highlight the correct exam



### 2. Select the correct processed images and manually transfer images to the desired Selected Network/PACS destination

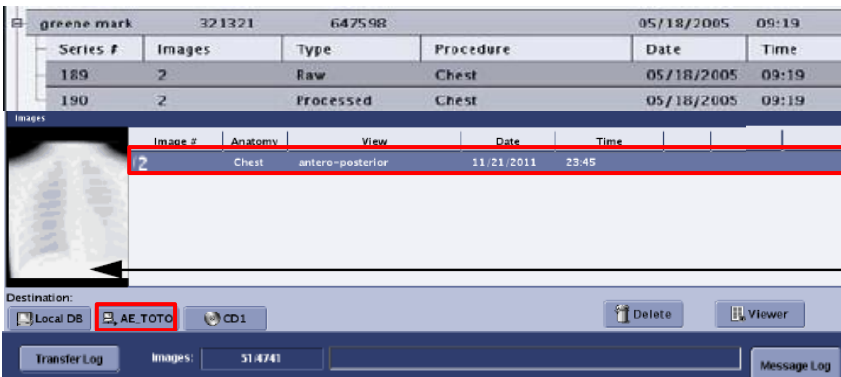

### 3. Ensure the correct images have transferred to the Selected Network/PACS destination by confirming at the Selected Network/PACS or system transfer log. Clicking on the transfer log icon the log will open display the current transfer status



← Transfer log icon

#### Transfer Log

| PatientID | Name     | Description | Submitted       | Source     | Destination  | Status               |
|-----------|----------|-------------|-----------------|------------|--------------|----------------------|
| 321321    | greene n |             | 17/08/04 09:... | DB_localdb | AR_c6devi... | Transfer in Progress |
| 321321    | greene n |             | 17/08/04 09:... | DB_localdb | AR_c6devi... | Completed            |

← Transfer in Progress

← Transfer Completed